# VIR-ELISA ANTI - TOXO- IgG

## 1. INTENDED USE

Enzyme immunoassay for quantitative determination of IgG antibodies to *Toxoplasma gondii* in human serum and plasma.

IgG [REF] EG 127 ▼ 96 [IVD]

## 2. PRINCIPLE OF THE ASSAY

The detection of antibodies is based on the principle of an enzyme-linked immunosorbent assay (ELISA). The purified, homogeneous antigen is fixed to each well of the microtiterstrips. Any specific antibodies present in the patient's sample are bound during the first incubation. After removing unbound material by washing, the presence of specific antibodies is detected using Anti-human IgG conjugate during the second incubation.

Excess peroxidase conjugate is then removed and TMB substrate is added, resulting in the development of a blue colour. The enzyme reaction is terminated by the addition of a stop solution. The intensity of the yellow colour thus developed is proportional to the concentration of antibodies in the sample.

## 3. DIAGNOSTIC RELEVANCE AND INTERPRETATION OF RESULTS

In the diagnosis of Toxoplasmosis a test combination is recommended to differentiate between acute and past infection.

- The detection of **specific IgG antibodies** is positive 1 or 2 weeks after infection. The IgG titre reaches a maximum after a few weeks, then falls in the course of the following months, generally remaining at a low level lifelong.
- The determination of **IgM antibodies** is an important indicator of an acute infection, or congenital toxoplasmosis. As the first immune response to infection the IgM antibodies appear after a few days and disappear generally after 3-5 months.
- But IgM antibodies can persist for some years after infection so further tests are necessary to clarify the stage of the infection.
- The presence of IgA antibodies indicates an acute infection. IgA antibodies to *Toxoplasma gondii* occur 10-30 days after an acute infection, somewhat later than IgM antibodies, and disappear after 3-6 months.

IgG	IgM	Interpretation	Recommendation
-	-	No specific antibodies detectable	During pregnancy recheck in the 2nd and 3rd trimester.
-	+	Possible early phase of infection, probable recent infection, reinfection or latent infection may also be possible.	<b>Monitoring</b> of IgG, IgM antibodies (Sample collection within 10-14 days) to determine seroconversion; supplementary tests e.g. IFA, CF, ISAGA, immunoblot, determination of Toxo specific IgA antibodies and determination of the avidity of the Toxo specific IgG antibodies.
+	-	Possible past infection, probable congenital toxoplasmosis, or reinfection	By suspect of acute infection – <b>monitoring of IgG- and IgM-antibodies.</b> supplementary tests e.g. IFA, CF, ISAGA, immunoblot, determination of Toxo specific IgA antibodies and determination of the avidity of the Toxo specific IgG antibodies.

- negative, + positive

## 4. PERFORMANCE CHARACTERISTICS

### Specificity / Sensitivity

213 samples were tested in parallel in VIR-ELISA ANTI-TOXO-IgG and comparison methods (IFA and ELISA). The specificity and sensitivity are based on the results found.

**Specificity:** IgG 99,1%      **Sensitivity:** IgG 100%

For the calculation of the **specificity and sensitivity** of the ELISA test, equivocal results were defined as positive results. The results refer to the groups of samples investigated.

## Precision and reproducibility

Intra-assay reproducibility was determined by testing samples of different levels of antibody reactivity for at least 22 times in one test run. The coefficient of variation (CV) of the reactive IgG samples was < 10%.

Inter-assay reproducibility was determined by testing samples of different levels of antibody reactivity in 10 different test runs. The CV of the reactive IgG samples was < 10%.

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## 6. KIT COMPONENTS

### 1. MICROTITERSTRIPS [MTS]

One microtiterplate is supplied which contains 12 microtiterstrips of 8 breakapart wells. The wells are coated with purified, inactive antigen. Strips are colour-coded.

### 2. PEROXIDASE CONJUGATE [CONJ|POD|IgG]

One vial containing 12 ml of ready-to-use sheep anti-human IgG Peroxidase conjugate. Peroxidase conjugate contains 0.049% Thimerosal as preservative.

### 3. NEGATIVE CONTROL [CONTROL-]

One vial of 1.2 ml containing human serum with 0,095% sodium azide as preservative. Ready to use.

### 4. CALIBRATOR 1 [CAL 1]

One vial of 1.2 ml containing human serum with 0,095% sodium azide as preservative. Ready to use. The unit value (I.U./ml) is reported on the label.

### 5. CALIBRATOR 2 [CAL 2]

One vial of 1.2 ml containing human serum with 0,095% sodium azide as preservative. Ready to use. The unit value (I.U./ml) is reported on the label.

### 6. CALIBRATOR 3 [CAL 3]

One vial of 1.2 ml containing human serum with 0,095% sodium azide as preservative. Ready to use. The unit value (I.U./ml) is reported on the label.

### 7. CALIBRATOR 4 [CAL 4]

One vial of 1.2 ml containing human serum with 0,095% sodium azide as preservative. Ready to use. The unit value (I.U./ml) is reported on the label.

### 8. TMB SUBSTRATE [SUBS|TMB]

One vial containing 13 ml of ready-to-use tetra-methylbenzidine (TMB) substrate.

### 9. SAMPLE DILUENT [SPE|DIL]

One bottle containing 100 ml (or 2x50 ml) of ready-to-use sample diluent buffer. The buffer includes 0.05% microbial preservative.

### 10. WASH SOLUTION 25X [WASH|BUF|25X]

One bottle containing 80 ml (or 2x40 ml) of wash solution concentrate.

### 11. STOP SOLUTION [SOLN|STOP]

One bottle containing 15 ml of 0.95N H<sub>2</sub>SO<sub>4</sub> stop solution. Ready-to-use.

### 12. ADHESIVE SEALS

*The safety data sheet ( MSDS) is available upon request.*

## 7. STORAGE AND STABILITY

Store all reagents at 2-8°C. Protect them from intense light and do not freeze. The expiration date of each component is indicated on the respective vial label. Do not use reagents beyond the expiration date.

After opening, [MTS] must be stored at 2-8°C in the plastic bag with desiccant and are stable up to 4 weeks.

The diluted [WASHBUF] is stable up to 4 weeks if stored at 2-8°C.

Use only [MTS] with an intact vacuum packaging.

## 8. MATERIALS AND EQUIPMENT REQUIRED BUT NOT PROVIDED

- Test tubes for sample dilution
- Timer
- Micropipettes, multipipettes 10-1000 µl
- One-litre graduated cylinder, distilled water
- ELISA washer or multichannel pipette
- Spectrophotometer for micro-plates (450-nm/ reference wavelength 630/620 nm)
- Paper towels, pipette tips
- Incubator for  $\lambda$  37°C with high relative humidity

## 9. WARNINGS OR PRECAUTIONS SAFETY PRECAUTIONS

The ELISA test is for **IVD** use only.

- Do not mix lot specific reagents, such as **MTS**, **CONTROL** / calibrators and **CONJPOD** from different kit lots. The **SUBSTMB** must not be from the original test kit, but the lot of the **SUBSTMB** must be the same as indicated on the kit label. The **SPEIDIL** (except Immunocapture assays), the **WASHBUF25x** and the **SOLNSTOP** can be used for all ELISA tests.
- Seal all bottles properly after use in order to avoid bacterial contamination. All samples and kit components should be considered potentially infectious. The **CONTROL** / calibrators have been tested for Hepatitis Bs antigen, anti-HIV I, anti-HIV II, anti-HCV (CE/FDA), and found to be negative.
- The **MTS** are coated with inactive antigen. However, normal laboratory precautions should be maintained when handling with infectious material. Do not pipette by mouth.
- Avoid contact with skin and mucous membranes when handling reagents, which contain preservatives (see kit contents). Wash thoroughly with water in case of contact and possibly look up a doctor.
- CONTROL** / calibrators containing sodium azide may react with lead and copper plumbing, building up explosive metal acids. Flush with sufficient water when disposing of reagents.
- The **SOLNSTOP** (0,95 N H<sub>2</sub>SO<sub>4</sub>) contains sodium hydroxide which may irritate skin and mucous membranes. Wash thoroughly with water in case of contact.
- For disposal the legal regulations have to be followed.

## 10. SPECIMEN COLLECTION AND STORAGE

- Only qualified and well-trained employees should carry out the assay procedure.
- The instruction for use describes the applicable test method. In case of modification or applications others than the intended use, or the use of automatic processors, the user has to validate the procedure and take the responsibility for it.
- Microbial contaminated specimen may cause interferences.
- Lipaemic, hemolytic or icteric samples should only be tested with reservations although in our testing no negative influence has been found.
- Suitable specimens are serum or plasma (heparinized, EDTA) samples obtained by standard laboratory techniques.
- The samples should not be heat-inactivated since non-specific results may occur.
- Results on tests using CSF are not available.
- Patient samples should be stored at  $\lambda$  2-8°C. For long-term storage  $\lambda$  -20°C or lower is recommended. Avoid repeated freeze-thaw cycles.
- Note:** Diluted patient samples must be used on the same day.

## 11. ASSAY PROCEDURE REAGENT PREPARATION

Bring all reagents to room temperature prior to use!

**WASHBUF:** Dilute the **WASHBUF25x** 1:25 with distilled water e.g. add 40ml of **WASHBUF25x** to 960 ml distilled water and mix well.

**Dilution of samples:** Dilute patient samples **1:101** with **SPEIDIL** e.g. 10µl sample + 1ml **SPEIDIL**, mix thoroughly.

**CONTROL** / Calibrators are ready to use.

**Note:** To take into consideration pipetting time, it is recommended to repeat the Calibrator 1 after every 6 strips (or after a pipetting time of  $\geq$ 7 min) to evaluate the following patient's tests with the new calculated cut-off value. In case of quantitative determination repeat calibrator 2 to 4 the same way Calibrator 1 was dispensed.

Take the required **MTS** out of the foil packets and place them in the holder. Possibly remaining wells of a **MTS** have to be stored at 2-8°C tightly sealed in the plastic bag provided, with the desiccant inside.

## 12. PIPETTING AND INCUBATION STEPS

- Pipette 100µl of the negative control, of calibrator 1 – 4 and of each diluted patient sample into the wells. Pipette 100 µl of sample diluent into well A1 (Blank).
- Cover the **MTS** with an adhesive seal and incubate the wells for 60 minutes at 37°C ( $\pm$  1°C) in an incubator with relative high humidity or in a humid chamber.
- Wash the wells four times as described in section k. **WASHING PROCEDURE**
- Add 100µl of ready-to-use peroxidase conjugate to each well.
- Cover the **MTS** with an adhesive seal and incubate the wells for 30 minutes at 37°C ( $\pm$  1°C) in an incubator with relative high humidity or in a humid chamber.
- Repeat washing as in section C above.
- Add 100µl of ready-to-use TMB substrate to each well.
- Incubate the wells at room temperature (18...25°C), in the dark for 15 minutes
- Add 100µl of stop solution to each well. Tap gently to ensure homogenous colour distribution and read within 10 minutes.
- To read the plate, make sure that the bottom is free from moisture and that no air bubbles are in the wells. Read the absorbance of the well contents at 450nm on a suitable plate reader. On readers equipped with a dual wavelength facility set the reference filter to 620/630 nm.

**Attention:** The absorbance (OD) of the Blank must always subtracted from the OD values of the negative control, the calibrators and samples.

### PROCEDURAL NOTES

Do not allow the wells to dry out between incubations. Comply with the given incubation temperatures and times.

### k. WASHING PROCEDURE

The washing procedure can be done manually with a multichannel pipette or on an automatic plate washer. Empty the wells, invert and tap dry on paper towel. Wash four times with a soaking time of approx. 30 seconds (300 µl).

## 13. SUGGESTIONS FOR TROUBLESHOOTING

In case that the ELISA instructions are followed strictly, the reagents are handled with care and the samples and reagents are pipetted carefully, the following kinds of errors can be avoided to a large extend.

ERROR	POSSIBLE CAUSES
No colorimetric reaction after addition TMB substrate	No Peroxidase conjugate dispensed, contamination of Peroxidase conjugate (possibly with control sera during pipetting) may cause an inactivation.
Generally too high reaction	Incorrect Peroxidase conjugate (i.e. not from original test kit), incubation time too long or incubation temperature too high, water quality for Washing Solution insufficient (low grade of deionisation)
Generally too weak reaction	Incorrect Peroxidase conjugate (i.e. not from original test kit), incubation time too short, incubation temperature too low
Reagent blank too high	Incorrect pipetting of sample diluent, contaminated reagents, reagents expired, exceeding of incubation time and temperature, external contamination of the bottom of microtiterstrips, (clean carefully!)
False positive / negative samples	Incorrect dilution of samples, microbial contaminated specimen
Unexplainable outliers	Contamination of pipettes, tips or containers or with metals (iron, copper etc.), insufficient washing
High variation (within a series)	Reagents (including microtiterstrips) not pre-warmed to room temperature prior to use. Washer is not washing correctly!
High variation (from series to series)	Incubation conditions not constant (time, temperature) high variation of incubation temperature, controls and samples are not carried out at same time (same intervals) check pipetting order, person related variation, strips dried out after washing (results are not reproducible).

## 14. VALIDITY OF THE ASSAY

**CONTROL** / calibrators should be carried out with every test run.

The test must comply with the following validation criteria:

- OD-value of the Negative Control should be  $< 0.100$ ,
- OD-value of Calibrator 1 should be  $> 0.200$ ,
- Ratio of Calibrator 4 / Cut-off value should be  $\geq 3.5$
- OD-value of the Blank should not be higher than 0.100.

If **CONTROL** /calibrators give invalid levels then results from the test samples are invalid too and retesting is required.

## 15. CALCULATION OF RESULTS

### A QUALITATIVE CALCULATION

Calculation of "Cut-off Value"

CUT-OFF VALUE = OD VALUE OF THE **CAL1**

CUT-OFF RANGE = CUT-OFF VALUE  $\pm 10\%$

Interpretation of sample results:

RESULT	DEFINITION
negative -	OD value sample $<$ Cut-off value -10%
equivocal	OD value sample $\geq$ Cut-off value -10% OD value sample $\leq$ Cut-off value +10%
positive +	OD value sample $>$ Cut-off value +10%

Equivocal results should be retested. Following the confirmation of the equivocal result the monitoring of the patient's antibodies is recommended in order to exclude unspecific reactions, which may also cause equivocal results.

### B CALCULATION OF RATIO (CUTOFF INDEX, COI):

Patient samples may also be quantified and interpreted by means of the calculation of the ratio (Cutoff Index, COI):  
COI = OD value of sample/ Cut-off value,  
whereby a ratio of 1.000 is equivalent to the Cut-off value.

Interpretation of sample results:

**Ratio  $< 0,9$**  negative result  
**Ratio  $0,9-1,1$**  equivocal result  
**Ratio  $> 1,1$**  positive result

### C CALCULATION OF INTERNATIONAL UNITS (I.U.) QUANTITATIVE

A quantitative diagram is enclosed. The first point of the curve is obtained using the **Cut-off value** (y-axis) and the I.U. value given on the **CAL1** label as appropriate (x-axis). The remaining points of the curve are obtained from the absorbance of the **CAL2**, **CAL3**, **CAL4** (y-axis) and their I.U. values (x-axis) as indicated on the label. The **Standard curve** is produced by drawing a **point to point** curve between the obtained values of the calibrators. The I. U. (International Units) of the patient samples may be read from the curve. The graph is at least linear up to the I.U. of **CAL4**.

**Note:** Samples higher than **CAL4** should be diluted further with **SPE/DIL** according to the expected value.

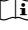
For the calculation of the results the dilution factor has to be considered.

The quantitative test results (I.U.) are based on the the WHO "International Standard for ANTI-TOXOPLASMA Serum (3<sup>rd</sup> International Standard Preparation)".

Regarding diagnostic relevance and interpretation of results see **page 1**.

For further information please visit our website:  
<http://www.viro-immun.de/>

## Symbole nach IVD/ symbols used with IVD devices/ Symbole / Símbolos/ Simboli/ Simbolos/ Symboly/ Címkékén/ Συμβολα IVD

<b>MTS</b>	Mikrotiterstreifen/ microtiterstrips/ Microplaques sensibilisées/ placa de microtítulo/ piasta microtitolo/ placa do microtitre/ Mikrotitračni Stripy/ Mikrotitercsíkok/ Ταινίες μικροπιλοποίησης
<b>SPE/DIL</b>	Probenverdünnungspuffer/ dilution buffer/ Tampon de dilution/ reactivo compensador/ soluzione tampone/ estabilizador de diluição/ Reditido na Vzorek/ Mintahígító/ Αραιωτικό Δείγματος
<b>WASHBUF25x</b>	Waschlösung/ wash solution/ solution de lavage/ solución limpiadora/ soluzione lavaggio/ solução de lavagem/ Konzentrat/ concentrate / Concentré / concentrado/ concentrato/ concentrado 25x/ Promývaci Roztok 25x/ Mosópufferkoncentrátum 25x/ Διάλυμα Πλύσης 25x
<b>CONJ/POD/IgG</b>	Peroxidase-Konjugat / Peroxidase conjugate / Conjugue Peroxidase/Conjugado Peroxidasa / coniugato con perossidasi/ conjugado Peroxidase/ Konjugát Peroxidáza/ Peroxidáz Konjugátum/ Συζευγμένη υπεροξειδάση IgG
<b>CONTROL-</b>	Negative Kontrolle/ negative control/ Contrôle négatif/ control negativo/ controllo negativo/ controle negativo / Negativní Kontrola/ Negatív Kontroll/ αρνητικός Μάρτυρας
<b>CAL1</b>	Calibrator/ calibrator/ calibreteur/ calibrador/ calibratori/ calibrador/ Kalibrátor/ Kalibrátorok/ Διάλυμα Βαθμονόμησης 1
<b>CAL2</b>	Calibrator/ calibrator/ calibreteur/ calibrador/ calibratori/ calibrador/ Kalibrátor/ Kalibrátorok/ Διάλυμα Βαθμονόμησης 2
<b>CAL3</b>	Calibrator/ calibrator/ calibreteur/ calibrador/ calibratori/ calibrador/ Kalibrátor/ Kalibrátorok/ Διάλυμα Βαθμονόμησης 3
<b>CAL4</b>	Calibrator/ calibrator/ calibreteur/ calibrador/ calibratori/ calibrador/ Kalibrátor/ Kalibrátorok/ Διάλυμα Βαθμονόμησης 4
<b>SUBS/TMB</b>	TMB-Substrat/ TMB substrate/ substrat TMB/ substrato TMB / Substrát TMB/ TMB Szubsztrát/ Υπόστρωμα TMB
<b>SOLN/STOP</b>	Stopplösung/ stop solution/ Solution d'arrêt/ solución de parada/ soluzione d'arresto/ solução de parada/ Stop Činidlo/ Stop Oldat/ Διάλυμα Τερματισμού
<b>Bib</b>	Literatur/ Literature/ Littérature/ Bibliografie/ Bibliografia/ Literatura/ Irodalom/ Βιβλιογραφία
<b>LOT</b>	Charge/ lot/ Lot/ lote/ carcia/ lote/ Číslo Šarže/ Lot Szám/ Αριθμός παρτίδας
<b>IVD</b>	In-vitro-Diagnostikum/ in vitro diagnostic/ Diagnostic in vitro/ diagnóstico in-vitro/ In-vitro diagnostic/ diagnóstico In-vitro/ In vitro Diagnostický Zdravotnický Prostředek/In Vitro Diagnosztikum/ Διαγνωστική ιατρική συσκευή In vitro
<b>REF</b>	Artikel Nr./ reference or order number/ Référence ou numéro de commande/ referencia o número de pedido/ codice di riferimento o di commissione/ referência ou número de encomenda/ Katalógové Číslo/ Katalógusban Szereplő Kód/ Κωδικός Καταλόγου
 96	96 Bestimmungen/ tests/ testés/ determinazioni/ testes / Počet Testů/ Vizsgálatok Száma/ Αριθμός εξετάσεων
	Gebrauchsanweisung beachten/ consult instructions for use/ consulter le mode d'emploi/consultar las instrucciones de uso/ consultare le istruzioni per l'uso/ consultar instruções de uso/ Přečtěte si Návod k Použití/ Olvassa el a Használati Utasítást/ Δείτε Οδηγίες Χρήσεως
	Temperaturgrenzen/ temperature limitation/ Limites de température/ Limites de temperatura/ Limiti di temperatura/ Limites de temperatura/ Teplotní Omezení/ Hőmérsékleti Korlátozások/ Θερμοκρασιακά όρια
	Verfallsdatum/ expiry date/ date d'expiration/ Fecha de caducidad/ Data di decadenza/ Limite de validade/ Datum Expirace/ Lejárati Idő/ Ημερομηνία λήξης (Χρήση έως ...)
	Hergestellt von/ manufactured from/ fabriqué par/ elaborado por/ fabbricato da/ produzido por/ Výrobce/ Gyártó/ Κατασκευάζεται από...

**Viro-Immune Labor-Diagnostika GmbH**, In der Au 29, D-61440 Oberursel, Germany -Tel.: +49-6171-6281-00 - Fax: +49-6171-6281-12  
email: info@viro-immun.de

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