

ravo SOX1/Titin Line Assay
Recombinant Line Assay for the Detection of
Autoantibodies to SOX1 and Titin

8, 16 or 24 Determinations

Product-No: MYAS1-001 (8 Determinations)
MYAS1-002 (16 Determinations)
MYAS1-003 (24 Determinations)

Manufacturer: **ravo** Diagnostika GmbH
Oltmannsstrasse 5
D-79100 Freiburg, FRG
Tel.: +49-(0)761-40 74 88
Fax: +49-(0)761-40 74 77
E-mail: info@ravo.de
www.ravo.de

Summary:

SOX1: In up to 50 % of patients with Lambert-Eaton myasthenic syndrome (LEMS) - a disorder mediated by voltage-gated calcium channels (VGCC) - cancer is detected, almost always a small cell lung cancer (SCLC). Recently it has been shown that in 43 % of patients with LEMS and SCLC the detectable antibodies (AGNA = anti-glial nuclear antibodies) are directed to SOX1. SOX1 is a protein which plays a role in neuronal development and which is expressed in SCLC¹. In single cases autoantibodies to SOX1 have been detected in patients with paraneoplastic limbic encephalitis. Antibodies to SOX1 can also be detected in other paraneoplastic neurological syndromes in association with the well characterized antineuronal autoantibodies.

Titin: Autoantibodies to Titin, the giant elastic intracellular protein of the striated muscle are detected in 70 -90 % of patients with Myasthenia gravis with an underlying thymom. The immune response is directed to a 30 kd (= MGT30 = myasthenia gravis/thymoma-specific 30-kd antigen) main immunogenic region (= MIR). Autoantibodies to Titin correlate with the severity of the disease. Anti-Titin Autoantibodies belong to the group of “partially characterized” antineuronal antibodies².

Principle:

Nitrocellulose strips which are coated with highly purified recombinant SOX1 and Titin (MGT30) are incubated with a specimen of patient serum. Specific antibodies in the specimen will bind to the antigen. Non specific molecules in serum specimens will be removed by washing the strips. Bound antibodies are detected by alkaline phosphate conjugated anti-human IgG using BCIP/NBT as substrate.

Storage:

All kit components and unopened opened are stable until date of expiry stored at +2....8°C. Diluted wash buffer is stable for 4 weeks stored at +2 to +8°C.

Specimen:

Preferably freshly collected specimen (serum or plasma) should be used. Specimen may be stored at 2 – 8°C for up to 5 days. For longer storage at 2 – 8°C antimicrobial agents (e.g. Thimerosal at a final dilution of 0.01%, ProClin300 at a final dilution of 0.03% or NaN₃ at a final dilution of 0.09%) should be added.

For long term storage (several months) specimen should be stored in aliquots at – 20°C. Avoid repeated freeze thawing.

Contaminated specimen may lead to false positive or negative results and should not be used. Some serum samples show a more or less strong violet background due to serum components of unknown origin. Such samples should be retested using another method e.g. IFA.

Reagents:

<u>Component</u>	<u>Amount/Volume</u>	<u>Description</u>
Nitrocellulose-Strips	8 (MYAS1-001) 16 (MYAS1-002) 24 (MYAS1-003)	Nitrocellulose-Strips for the detection of IgG antibodies, black screw cap
Wash buffer	1 x 50 ml (MYAS1-001) 2 x 50 ml (MYAS1-002) 3 x 50 ml (MYAS1-003)	Wash buffer, 20 x concentrated, blue screw cap
Dilution buffer for samples	1 x 25 ml (MYAS1-001) 1 x 25 ml (MYAS1-002) 2 x 25 ml (MYAS1-003)	ready to use, contains 0,03% ProClin300, green screw cap
Conjugate for samples and positive control	1 x 20 ml (MYAS1-001) 2 x 20 ml (MYAS1-002) 3 x 20 ml (MYAS1-003)	ready to use, Alkaline Phosphatase Conjugate contains 0,03% ProClin300, red screw cap
Substrate	1 x 20 ml (MYAS1-001) 2 x 20 ml (MYAS1-002) 3 x 20 ml (MYAS1-003)	BCIP/NBT ready to use, black screw cap
Incubation trays	1 x (MYAS1-001) 2 x (MYAS1-002) 3 x (MYAS1-003)	
Instruction for use	1 x	

Additional reagents available on request:

Positive Control	2 ml	ready to use, contains 0,03% ProClin300, red screw cap
Product No.:	MYAS1-PKO	
Negative Control	2 ml	ready to use, contains 0,03% ProClin300, colourless screw cap
Product No.:	PNSNKO	

Reconstitution:

- Make sure all kit components are at room temperature before use.
- Dilute the wash buffer concentrate 1:20 with distilled water. **During storage at low temperatures, crystals may form in concentrated wash buffers, which can be dissolved by incubating the concentrate at 37°C for 30 minutes.** Let the solution cool down again to room temperature before use. Diluted wash buffer is stable for 4 weeks stored at +2 to +8°C.

Procedure:

The Nitrocellulose strips are labeled at the bottom (if not indicated otherwise). They must be incubated with the labels facing upwards and should be completely covered with fluid during all incubation steps.

**Cover the other strips with 2 ml dilution buffer. Add 10 µl of the specimen and mix carefully (end-dilution: 1 : 200)
Incubate for 30 minutes at room temperature on a rocking table.**

- wash with diluted wash buffer: Carefully remove the fluid using a pipet or pour away the fluid from each strip. Add ca. 2 ml of diluted wash buffer to each strip and shake for ca. 30 seconds. Repeat five times.
- Add 2 ml alkaline phosphatase IgG conjugate, ready to use, per strip.

Incubate for 30 minutes at room temperature on a rocking table.

- wash with diluted wash buffer: Carefully remove the fluid using a pipet or pour away the fluid from each strip. Add ca. 2 ml of diluted wash buffer to each strip and shake for ca. 30 seconds. Repeat five times.
- Incubate each strip in 2 ml ready to use substrate-solution.

Incubate for 20 minutes at room temperature until the bands become clearly visible. See control scan for comparison.

- Transfer the strips to distilled water to stop the reaction. Put the strips onto filter paper and let them dry. Store the strips in the dark.

Interpretation:

Put the strips side by side to assign the proteins. Only strips from the same batch should be compared.

The positive control and control scan helps to assign the proteins, the documentation sheet to stick the strips on the latter for documentation.

In contrast to the well-known paraneoplastic autoantibodies low antibodies titers to SOX1 are sometimes detected. The meaning of these weak reactions is unknown at the moment.

Remarks:

- Avoid contact of the skin with substrate solution.

References:

1. L.Sabater, M.Titulaer, A.Saiz, J.Verschuuren, A.O.Güre and F.Graus. SOX1 antibodies are markers of paraneoplastic Lambert-Eaton myasthenic syndrome. *Neurology*, 2008; 70:924-928.
2. E.Lübke, A.Freiburg, G.O.Skeie, B.Kolmerer, S.Labeit, J.A.Aarli, N.E.Gilhus, R.Wollmann, M.Wussling, J.C.Rüegg and W.A.Linke. Striational autoantibodies in myasthenia gravis patients recognize I-band titin epitopes. *J Neuroimmunol*, 1998; 81:98-108