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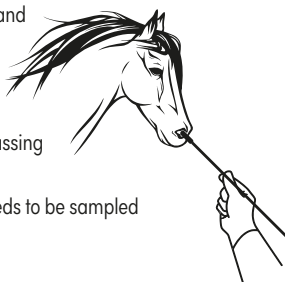
PROCÉDURE

01

Collecting specimen : the swab should be passed into the ventral meatus of the horse's nasal passage until the nasopharynx. Rotate the swab 5 times to the left and right to pick up and absorb respiratory secretions.


Make sure to avoid the false nostril in passing the swab.

*Only one nostril needs to be sampled


02

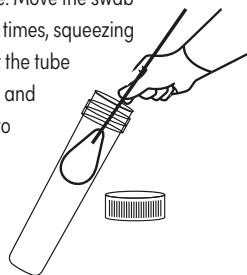Slowly insert the swab head into the Transport Medium tube. Move the swab **slowly** up and down 10 times, squeezing the swab head against the tube side. Remove the swab and screw the cap back onto the tube.

03

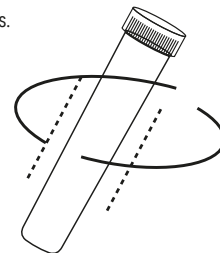Gently mix the Transport Medium tube upside down a few times.


04

Remove the qPCR tubes strip from the pouch.

05

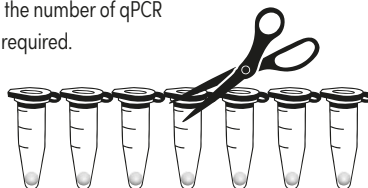Select the number of qPCR tubes required.


06

Gently tap the qPCR tube on a solid surface check that the white reagent beads are at the bottom of each qPCR tube and they appear identical in size.


07

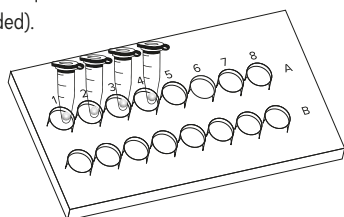Place the qPCR tubes in the rack (provided).

08

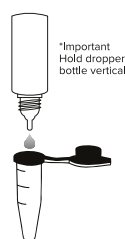
Open the qPCR tube(s), add one drop from the Distilled Water Dropper Bottle (blue cap).

*Important
Hold dropper
bottle vertical

09

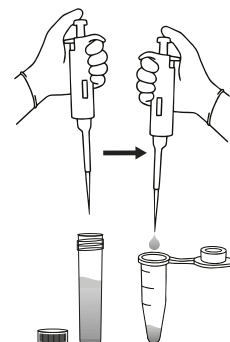Unscrew the Transport Medium tube, take out sample with the pipette and add sample into the qPCR tube, Gently mix, avoid bubbles in the qPCR tube.


10

Check PCR tubes and check that each tube has an equal volume of fluid. The samples are now ready to be analysed by the InnoScreen G8.

11

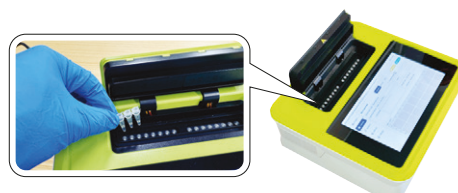On the InnoScreen G8 analyser
Follow the instructions to analyse the samples.


Press to open door button on InnoScreen G8 to open the lid and follow the instructions to analyse samples.

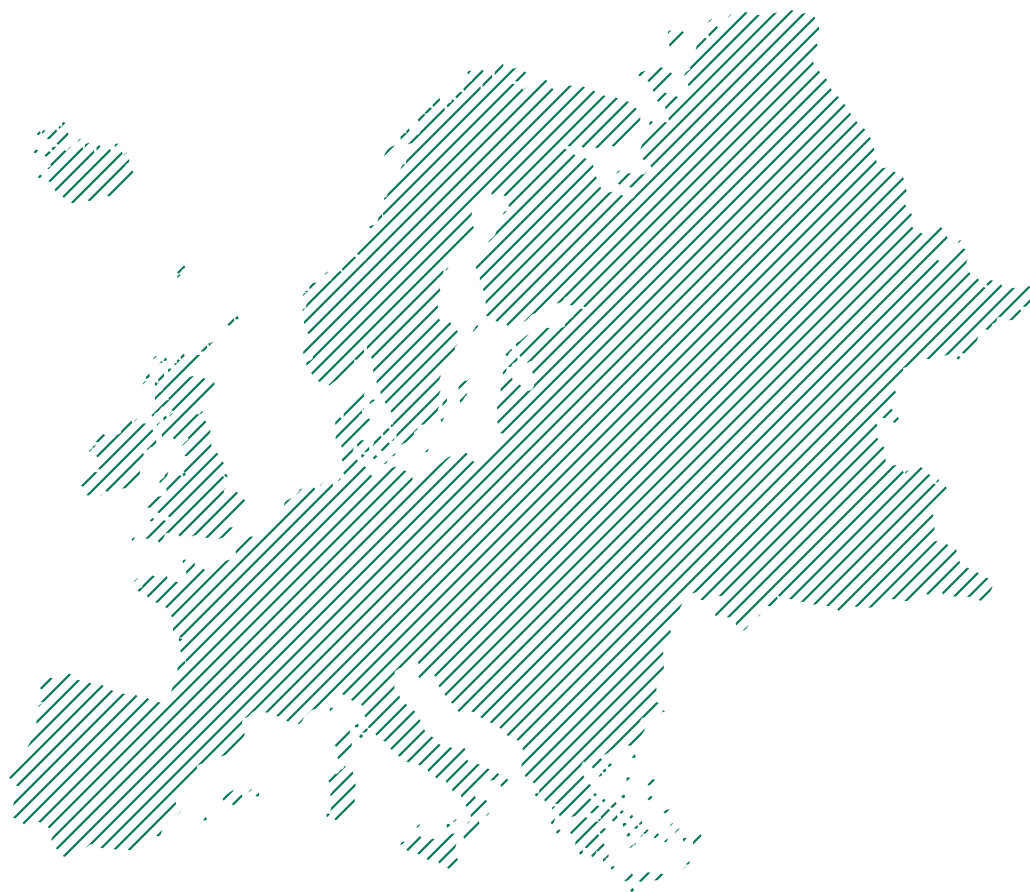Result:		
Streptococcus equi equi	(Ct: none)	
NEGATIVE		
Streptococcus equi zooepidemicus	(Ct: none)	
NEGATIVE		
Equine Herpesvirus-1	(Ct: 30)	
POSITIVE		
Equine Herpesvirus-4	(Ct: none)	
NEGATIVE		
Equine Influenza	(Ct: none)	
NEGATIVE		

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